Thank you for your purchase of Naboso insoles !

Features

Patent-pending Texture

All of the Naboso insoles are uniquely designed with a two-point discrimination texture which stimulates the nerves in the bottom of the feet. This unique texture signals the brain to where the foot is when standing and walking. The more sensory information the brain has, the better the motor control and movement patterns.

Super thin design

All the Naboso insoles are designed to be only 2-3 mm in thickness. This minimal design allows the Naboso insoles to fit into almost every shoes including dress shoes, cleats and ballet flats. If additional support is needed, the Naboso insoles are thin enough to be placed directly on top of any custom orthotic or arch support.

Freedom of movement

There is intentionally no arch or biomechanical support provided by the Naboso insoles. Our unique insole allows full movement of the foot which makes it the perfect addition to your favorite minimal shoes. The flexibility of the Naboso insoles also allows the insole to mold to your unique foot shape or mold to an arch support or custom orthotic.

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How it works

Step 1 :

Receptors on the bottom of the feet sense stimulation including texture, pressure and vibration.

Step 2 :

Standing on the Naboso insoles stimulates the nerves that are sensitive to texture.

Step 3 :

The region of the brain responsible for the movements is stimulated.

Step 4 :

Feel the improved balance, stability and body awareness.

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Pick any shoes

Step 2 :

Remove the existing shoes insole, place over the Naboso insole and trace the outline onto the Naboso Insole

Step 3 :

Carefully cut away any excess material from the Naboso insole.

Step 4 :

Slip the Naboso insole into the shoe with or without the original insole. Based on comfort, the Naboso insoles can go on top of any existing insole or orthotic.

Alternating between textures

After wearing the Naboso insoles you may begin to adapt to the texture – this is normal ! To help avoid this adaptation we recommend alternating between two of the Naboso textures. You can also skip a day in between Naboso Insole use. Check out our Naboso Bundle Packages which include all textures.

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How to clean

Step 1 :

Remove the Naboso insole from your shoes

Step 2 :

Place the Naboso insole into the sink

Step 3 :

Clean with antibacterial soap, water and a brush

Step 4 :

Allow to air dry for 30 minutes

Naboso cleaning guidelines

Naboso insoles are machine washable. We recommend cleaning your Naboso Insoles every 1 – 2 weeks to help eliminate odor and increase the lifespan of the insoles. Due to their unique textural design the Naboso Insoles can exfoliate the feet which contributes to odor. If you notice any odor make sure to use an anti-bacterial soap or tea tree oil.

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